


# Challenger Canopy (J620)

**NOTE:** The J620 Canopy requires that your mobility device has a 1" square, Accessory Receiver, behind and below the seat.

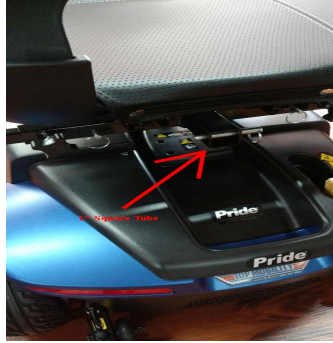

## Included Items:

- a. Clevis Pins (2) b. Locking Knob (3) c. Canopy Frame d. Middle Bracket e. Lower Bracket f. Canopy Cover



The Lower Mounting Bar May not be shaped exactly like in this picture)

1. Stretch Canopy Cover onto Canopy Frame.
  - 1a. The small pocket on the Canopy Cover goes to the front and underside of the Canopy Frame.
  - 1b. Secure 3 Velcro straps to Canopy Frame.
  - 1c. Secure Velcro on front of cover, inside the pocket.
2. Attach the middle Bracket (T-Shaped Bracket) to the Canopy Frame.
  - 2a. Attach using two (2) Locking Knobs.
  - 2b. Pull Velcro straps taught and secure around Middle Bracket frame.
3. Insert Lower Bracket.
  - 3a. Insert the short end of the Lower Bracket into Receiver under the seat.

**NOTE:** There are several holes in the Lower Bracket and the Receiver, so you can adjust how far the J620 Canopy extends backwards from the seat.

- 3b. Insert Clevis Pin. If there is a locking knob on the Receiver, tighten the locking knob.

4. Insert Middle Bracket into Lower Bracket
  - 4a. Insert into the open end of the Lower Bracket.

**NOTE:** There are several holes so you can adjust how high it extends above your head.

- 4b. Insert Clevis Pin. Tighten the Locking Knob.

5. The J620 Canopy is ready for use.