

# JCH10 Cane Holder

## Items included

- a. Cane Holder



Figure 1



Figure 2

**Assembly** - No Assembly Required

## Mounting

1. Remove Cane Holder from protective wrapping.
2. There are 4 straps on the cane holder that will need to be adjusted. Two of the straps (A & B) are identical and perform the same function, just a couple inches apart. (Figure 2)
3. Loosen Strap-C to its fullest length. Slip it over the top of the seat and move it down about half-way the seat back. Do not tighten until Straps A & B are adjusted.

# JCH10 Cane Holder

4. Unlace the two straps (A & B) at the top of the cane holder. Slip straps over and under seat back. Figures 3.

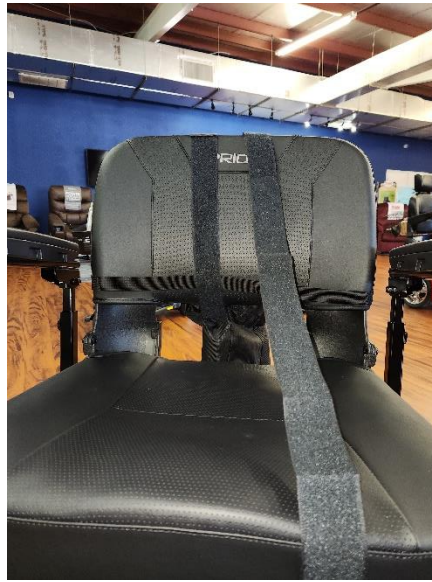


Figure 3

5. Pull straps tight, but not too tight as to crease seat back.
6. Once Straps A & B are extended long enough to wrap around the seat back, re-lace them into the belt loops they were taken out of. Velcro is there so the adjustments are not lost upon re-lacing of the straps. You can tighten Strap-C to fit at this time.
7. Un-Velcro Strap-D from itself. Loop Strap-D around the seat post under the seat. Do not pull it too tight as to bend or distort the way the cane holder hangs. Re-attach the Velcro.

NOTE: If you wish to use your 1" Square Receiver under the seat, simply redo this process off to one side a little.