



Translator: Rollator and Transport Chair - 2 in 1

300 LBS
WEIGHT CAPACITY

AVAILABLE COLORS:



USER MANUAL - 800 TRANSLATOR ROLLATOR

GENERAL EXPLANATION OF YOUR TRANSLATOR



Your walker is equipped with various components and parts. Learn this before you proceed with studying this manual. Designs and specifications may change without prior notice.

SAFETY PRECAUTIONS

- Do not allow children to play with packaging without supervision. Risk of suffocation for children!
- Keep the product away from children, it is not a toy. Children may tip over and injure themselves.
- The walker should only be used for what has been developed and make sure that the walker works well before use. All screws should be tightened.
- Use the walker only on flat, smooth surface. Avoid sewers, grates or similar obstacles, as the wheels could get trapped.

Make sure that the walker is used only by someone who is familiar with the instructions above.

ASSEMBLY AND ADJUSTMENT

CAUTION!

Use the walker only when fully assembled and all components are secure. Do not use the walker if it has visible damage, or if there is a suspicion that it may be damaged!

BRAKE OPERATION

- Push up on brake handle lever to slow the rollator.
- Push brake handle lever down to lock the rear wheels.
- Pull up on brake lever handle to release the lock.



BRAKE ADJUSTMENT

- Turn the adjuster screw anti-clockwise to increase the tension then lock the adjustment by turning the locking nut clockwise while holding the adjuster screw still.

⚠ Reducing the brake pressure may effect the brake in the locked position.



ADJUSTMENT

- Adjust the height of walker to a comfortable position for the user.
- To adjust the height, loosen and remove knob screws on both sides and adjust the handle tubes to the desired height. Then tighten the knob screws until secure.

CARE AND MAINTENANCE

- Ensure that all attaching hardware is secure at all times.
- Replace any broken, damaged or worn parts immediately.

ASSEMBLY

1) Remove all packaging. The rollator will be folded as shown below.



2) Open up the main frame so the seat has locked into position as per the image below (you should feel both sides 'click' into position). Then place the main frame of the rollator on a solid level surface.



3) Insert the two height adjustable handle sections into the main frame marked "A" in Figure 1. Make sure the snap button has locked into position at your preferred height setting.

4) Insert the two rear brake wheels into the main frame marked "B" in Figure 1. Make sure the snap button has locked into position.

5) Insert the two front wheels into the main frame marked "C" in Figure 1, and then turn knob screws shown in the image below clockwise until it locks into position and the wheels are secure.



FIGURE 1

6) Attach the footrest provided onto the front of the frame, adjust the footrest height to desired position and secure with knob screws.



7) If necessary, attach the shopping bag in the front of the rollator, just beside the padded seat.



WARRANTY

Limited Lifetime Warranty is provided against any factory defects in material or workmanship of the walker frame.

One (1) year warranty against defective materials or workmanship on all other components, assemblies and parts.

This warranty does not cover device failure due to misuse, negligence, improper storage or handling, improper operation and unauthorized modifications.

Normal wear and tear on hand grips, brake cables, wheels, and totes are not covered under warranty. These items are subject to normal wear and need periodic replacement and/or adjustment. Please see your dealer for periodic evaluations of your walker.

It is recommended but not required that the walker be returned to the Lifestyle Mobility Aids dealer through whom it was originally purchased. However all Lifestyle Mobility Aids dealers are qualified to assist you in obtaining warranty service. If the walker is to be returned to Lifestyle Mobility Aids prior authorization will be required.

All Lifestyle Mobility Aids products are identified by a serial number at the time of manufacture. Removal or defacing the serial number may void this warranty. If you have a question about your Lifestyle Mobility Aids walker or this warranty please email us at contactus@lifestylemobilityaids.com.



Some Additional Products:



Commando Memory
Standard Crutch



Commando Memory
Forearm Crutch



Commando Memory
Cane

Designed In The USA and Distributed By:
LIFESTYLE MOBILITY AIDS
Email : contactus@lifestylemobilityaids.com
Website : www.lifestylemobilityaids.com