



Translator Deluxe

325 LBS

WEIGHT CAPACITY

- AMBER SUNRISE
- LASER BLUE
- LASER RED



900

USER MANUAL - TRANSLATOR DELUXE

GENERAL EXPLANATION OF YOUR WALKER



SAFETY PRECAUTIONS

⚠ Read entire manual prior to assembly and use. ⚠

We recommend you keep this User Manual in a safe place for future reference. Check contents of package and if parts are damaged or missing, contact your dealer immediately.

- Do not allow children to play with packaging due to risk of suffocation.
- Keep the product away from children, it is not a toy. Children may tip over and injure themselves.
- Use walker only on flat, smooth surfaces. Avoid sewers, grates or similar obstacles, as wheels could get trapped.
- Do not use the translator on any incline greater than 9 degrees, or on a slope with water, ice or snow covering.

TABLE OF CONTENTS

SAFETY PRECAUTIONS	2
SPECIFICATIONS	3
ASSEMBLY INSTRUCTIONS	3
OPERATING INSTRUCTIONS	4
CLEANING	4
REGULAR MAINTENANCE	5
WARRANTY	5

SPECIFICATIONS

NAME	Translator Rollator
UNIT DIMENSIONS	21" W x 31" L
HEIGHTS	31.5" - 37.5"
SEAT HEIGHT FROM FLOOR	19"
SEAT DIMENSIONS	15" W x 14" D
WEIGHT	19.5 lbs
WEIGHT CAPACITY	325 lbs

ASSEMBLY

- 1 Unfold Base Frame by flipping Sling Backrest (9) forward and pushing down on Seat (11) while holding Armrest (3).
- 2 Insert Push Handles (1) into Rear Frame (5) making sure brake cables are outside of frame and secure with Knob Screws (4) at desired height.
- 3 Install Footrests (15) by hooking onto Chrome Buttons (14) above front wheels. Attach to front of frame for use while seated or to back of frame when using as a walker. Footrests must be removed to fold Translator for storage.



USED AS TRANSPORT CHAIR



USED AS ROLLATOR

OPERATING INSTRUCTIONS

NOTE: Before use, check that all assembly steps were followed correctly and all components are secure.

Operating the Brakes

- 1 To lock, push down on brake levers until they click.
- 2 To unlock, pull up on brake lever.
- 3 When using as a rollator, gently squeeze brake levers until rollator stops.

NOTE: Brakes should always be locked before using as a seat or when walker is unattended.

Using as Transport Chair


- 1 Prepare Translator for User by flipping Sling Backrest to back of Translator and installing Footrests with plates flipped up. With Brakes locked, have User sit. Flip down Footplates so User can place feet on Footrests. Caregiver can then unlock Brakes and use Push Handles to transport user.

NOTE: Do not attempt to reach forward, backward or sideways to reach for objects when seated.

Using as a Rollator

- 1 Keep both hands on Push Handles with fingers around Brake Levers as needed to stop Translator.
- 2 Push forward to a comfortable arm's length and then step toward Translator and repeat.

CLEANING

 Do not use high pressure cleaners, steam cleaners or harsh, caustic chemicals for cleaning.

Frame

- › Wipe frame with a damp cloth.
- › For more stubborn dirt, use a mild detergent solution with warm water.
- › Dry thoroughly before use to avoid corrosion.
- › Clean the wheels using a damp brush with plastic bristles (do not use a wire brush or you may damage the rubber or painted finish).

Bag and Seat

- › Regularly clean the padding materials using a mild detergent solution and warm water, or wash with alcohol-based disinfectants if necessary.
- › Dry thoroughly before use.
- › Avoid contact with saltwater or pools with high levels of chlorination.

REGULAR MAINTENANCE

⚠ Professional inspection is recommended at least once a year.

⚠ Any service or repairs, especially to the brakes, must be performed by a trained professional. Any unauthorized repairs or service may void your warranty.

- › A risk assessment should always be carried out to ensure the product is suitable.
- › Before use, the user should read these instructions and familiarize themselves with the translator.
- › Ensure all components are properly and securely installed.
- › Avoid storing in direct sunlight; seat or parts exposed to sunlight will heat up and may cause injury to the skin.
- › Do not place more than 10 lbs of shopping or other items into the designated shopping bag.
- › Regularly check for damage and wear, paying special attention to the brakes and condition of the wheels. If the translator is damaged in any way, stop using the translator immediately and contact your authorized dealer or the manufacturer.
- › Check handle grips are secure and do not move.
- › Check the brake levers are firm; if loose or worn they must be replaced before use.
- › Check the efficiency of the braking system to ensure rear wheels do not move when the brakes are engaged or locked.

WARRANTY

Limited Lifetime Warranty is provided against any factory defects in material or workmanship of the walker frame.

One (1) year warranty against defective materials or workmanship on all other components, assemblies and parts.

This warranty does not cover device failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications.

Normal wear and tear on hand grips, brake cables, wheels, and totes are not covered under warranty. These items are subject to normal wear and need periodic replacement and/or adjustment. Please see your dealer for periodic evaluations of your walker.

It is recommended but not required that the walker be returned to the Lifestyle Mobility Aids dealer through whom it was originally purchased. However all Lifestyle Mobility Aids dealers are qualified to assist you in obtaining warranty service. If the walker is to be returned to Lifestyle Mobility Aids prior authorization will be required.

All Lifestyle Mobility Aids products are identified by a serial number at the time of manufacture. Removing or defacing the serial number may void this warranty. If you have a question about your Lifestyle Mobility Aids walker or this warrant please email us at contactus@lifestylemobilityaids.com.



Some Additional Products:



Premium Shower
Chair



Commando Memory
Forearm Crutch



Aluminum Companion
Chair

Designed In The USA and Distributed By:
LIFESTYLE MOBILITY AIDS
Email : contactus@lifestylemobilityaids.com
Website : www.lifestylemobilityaids.com