

CHALLENGER

MOBILITY

www.ChallengerMobility.com

ASSEMBLY INSTRUCTIONS


J900 MOBILITY BASKET

PARTS INCLUDED

a- T Shaped Bracket



b - J900 Basket

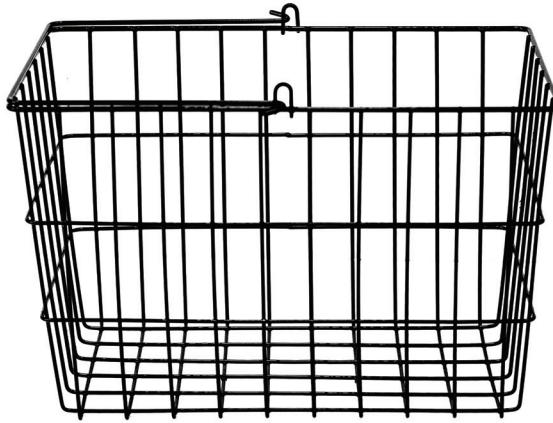

c - 4 Phillips



d - 2 Brackets



e - Clevis Pin



f- Locking knob

TOOLS REQUIRED: Phillips Screwdriver

ASSEMBLY INSTRUCTIONS

1. Place T shaped bracket (a) on work surface with screw holes facing up. Place basket (b) on top of T shaped bracket (a), parallel to cross bars with T brace close to edge of basket (a).
2. Align holes in 2 brackets (d) with holes in T shaped bracket (a), inside basket (b). Figure 1
3. Pre start 4 screws (c) through 2 brackets (d), through the bottom of the basket (b), into the T shaped bracket (a). Run 4 screws (d) almost all



the way down. A thread lock compound may be applied to the 4 screws (c) to prevent them from loosening. But is not necessary.

5. Tighten 4 screws (c) into T shaped bracket (a).

MOUNTING INSTRUCTIONS

1. Insert the end of the T bracket (a) into the accessory port on the back of the mobility scooter.
2. Align the holes on the side of the accessory port with the holes in the T bracket (a).
3. Insert clevis pin (e) in the side of the accessory port. Insert until clevis pin (e) protrudes from opposite side.
4. Insert locking knob (f) into threaded port on top of accessory port. Tighten down to lock basket in place (applies only on specific models).



CHALLENGER

MOBILITY



Scooter LED Light Mobility
Lockable Hard Saddlebags
Scooter Rear View Mirror Pair Cane
Trailer Scooter Locking Compartment
Extra Large Rear Basket for Mobility Scooter Scooter Cleaning Kit
Scooter backrest Bag Scooter Locking Compartment Medium Scooter Phone Holder **Deluxe**
Scooter Touch Up Paint Hybrid Ashtray or Cup Holder Scooter Umbrella Holder Assembly **Scooter**
Rain Poncho Large Rear Basket for Mobility Large **Deluxe Armrest Bag** Deluxe Armrest Bag
 Nonbreakable **Universal Scooter Cup** Scooter Decal Stickers **Scooter Safety Flag Assembly** Scooter
 Rear View Mirror Pair Side Mount **Scooter LED Safety Vest** Scooter Oxygen Tank Holder **Haul-N-Go**
 Scooter Tow Hitch Assembly Scooter LED Light Safety Alert Assembly Challenger X Batteries
 Challenger XL Scooter Carrier **Powerchair Joystick Weather Cover** Scooter Tiller Weather Cover

MOBILITY ACCESSORIES AND MORE

www.ChallengerMobility.com