Mega Scooter Backrest Bag Instructions

Tools Needed: Tape Measure

There are three sets of Velcro on your Mega Bag: 2 attached to the top of the bag, 2 attached to the back of the bag, and 2 adjustable straps with metal rings that can be detached. There are also 2 sets of nylon loops securing the Velcro on the back of the bag.

## **Instructions:**

- 1) Unfold your Mega Bag.
- 2) Pull apart the 2 Velcro straps at the top and release the adjustable Velcro straps using the metal rings.
- 3) Measure the adjustable Velcro strap from the bottom nylon loop on the back of your Mega Bag to the bottom of the metal ring. You want the adjustable Velcro strap to be 1" longer than the height of your backrest. Ex: If your backrest measures 16" from the seat, pull the adjustable Velcro strap 17" from the bottom nylon loop to the

Velcro side of the metal ring.

- 4) Any loose or excess adjustable Velcro strap can be cut or looped through the nylon loops.
- 5) At the top of the Mega Bag, fasten the adjustable Velcro strap to the attached Velcro strap using the metal rings.
- 6) Pull tightly to secure.